



Thrombin-Antithrombin Complex

Test ID: FFTAC

Explanation:

Effective May 4, 2026, Test ID FFTAC, referred to Esoterix Coagulation, will be obsolete and replaced with Miscellaneous LabCorp Test ID ZW76.

Recommended Alternative Test:

Miscellaneous LabCorp

Test ID: ZW76

Referral Lab Code: 015045

Test Name: Thrombin-Antithrombin Complex

Methodology:

Enzyme Immunoassay

Reference Values:

0.0 to 4.2 ng/dL

Specimen Requirements:

Collection Container/Tube: Blue top (3.2% buffered sodium citrate)

Submission Container/Tube: Plastic vial

Specimen Volume: 2 mL Plasma

Collection Instructions:

1. Do not draw from an arm with a heparin lock or heparinized catheter.
2. Draw blood in a blue-top tube containing 3.2% buffered sodium citrate.
3. Mix immediately by gently inverting at least 6 times.
4. Centrifuge, aliquot plasma into a plastic vial, being careful not to disturb the cells, and freeze immediately.
5. Send frozen.

Note: Critical frozen. Separate specimens must be submitted when multiple tests are ordered.

Specimen Minimum Volume

Plasma: 1 mL

Specimen Stability Information:

| Specimen Type | Temperature | Time |
|---------------|-------------|---------|
| Plasma Na Cit | Frozen | 28 days |

Fee:
\$195.70

CPT Code:
83520

Day(s) Setup: One time per week

Report Available: 5 to 13 days

Questions

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710.