

Thrombin-Antithrombin Complex

Explanation: The following new orderable, referred to LabCorp Burlington, will replace miscellaneous test ZW76/015045, Thrombin-Antithrombin Complex, effective May 18, 2026.

Test ID: FATCX

Methods:

Enzyme immunoassay

Reference Values:

0.0 to 4.2 ng/dL

Specimen Requirements:

Collection Container/Tube: Blue top (3.2% buffered sodium citrate)

Submission Container/Tube: Plastic vial

Specimen Volume: 2 mL Plasma

Collection Instructions:

1. Do not draw from an arm with a heparin lock or heparinized catheter.
2. Draw blood in a blue-top tube containing 3.2% buffered sodium citrate.
3. Mix immediately by gently inverting at least 6 times.
4. Centrifuge, aliquot plasma into a plastic vial, being careful not to disturb the cells, and freeze immediately.
5. Send frozen.

Note: Critical frozen. Separate specimens must be submitted when multiple tests are ordered.

Specimen Stability Information:

Specimen Type	Temperature	Time
Plasma Na Cit	Frozen	28 days

Fee:
\$195.70

CPT Code:
83520

Day(s) Setup: Once a week

Report Available: 5 to 13 days

Questions

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710