

Synthetic Cannabinoid Metabolites Screen, Expanded, Urine

Test ID: FCSC

Explanation:

Effective August 3, 2026, test ID FCSC, referred to NMS Labs, will become obsolete due to file definition changes.

Recommended Alternative Test:

Synthetic Cannabinoid Metabolites Screen-Expanded, Urine

Test ID: FSCBA

Reflex Tests:

Test ID	Reporting Name	Available Separately	Always Performed
FSCBB	Synthetic Cannabinoid Confirm, U	No	No

Testing Algorithm

If appropriate, Synthetic Cannabinoid Confirmation will be performed at no additional charge.

Methods:

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

Reference Values:

Reporting limit determined each analysis.

MDMB-4en-PINACA t-Butylacetic acid: None detected

MDMB-BUTINACA t-Butylacetic acid: None detected

5F-MDMB-PINACA t-Butylacetic acid: None detected

Specimen Requirements:

*****Must submit one specimen per order. Specimens cannot be shared between multiple orders.*****

Collection Container/Tube: Plastic, preservative-free urine container

Specimen Volume: 5 mL

Collection Instructions:

1. Collect 5 mL random urine specimen without preservative.
2. Send refrigerated in a plastic preservative-free urine container.

Minimum volume: 2.4 mL

Specimen Stability Information:

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	30 days
	Ambient	30 days
	Frozen	30 days

Fee: \$126.00**CPT Code:** 80307**Day(s) Performed:** Monday through Sunday**Report Available:** 10 to 16 days**Questions**

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710.