

ZnT8 Antibodies

Test ID: FZNT8

Explanation: Effective immediately, test ID FZNT8, referred to Esoterix Endocrinology, will reflect the following changes:

Current Specimen Required
Collection Container/Tube: Preferred: Red top Acceptable: Serum gel Submission Container/Tube: Plastic vial Specimen Volume: 0.5 mL Serum Collection Instructions: 1. Within 45 minutes of collection, centrifuge and aliquot serum into a plastic vial. 2. Send refrigerate.

New Specimen Required
Collection Container/Tube: Preferred: Red top Acceptable: Serum gel Submission Container/Tube: Plastic vial Specimen Volume: 0.5 mL Serum Collection Instructions: 1. Within 45 minutes of collection, centrifuge and aliquot serum into a plastic vial. 2. Send frozen.

Current Specimen Stability		
Specimen	Temperature	Time
Serum	Refrigerated (preferred)	7 days
	Ambient	7 days
	Frozen	64 days

New Specimen Stability		
Specimen	Temperature	Time
Serum	Frozen (preferred)	64 days
	Ambient	7 days
	Refrigerated	7 days

Questions:

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710.