

Leukotriene E4, 24 Hour, Urine

Test ID: TLTE4

Explanation: The following specimen and stability changes will occur on the effective date.

Current Specimen Information		
Ambient (no additive)	No	
Refrigerate (no additive)	Preferred	
Frozen (no additive)	OK	
50% Acetic Acid	OK	
Boric Acid	OK	
Diazolidinyl Urea	No	
6M Hydrochloric Acid	No	
6M Nitric Acid	No	
Sodium Carbonate	OK	
Toluene	No	

New Specimen Information		
Ambient (no additive)	No	
Refrigerate (no additive)	OK	
Frozen (no additive)	Preferred	
50% Acetic Acid	No	
Boric Acid	OK	
Diazolidinyl Urea	No	
6M Hydrochloric Acid	No	
6M Nitric Acid	No	
Sodium Carbonate	OK	
Toluene	No	

Current Specimen Stability		
Specimen	Temperature	Time
Urine	Frozen (preferred)	28 days
	Ambient	24 hours
	Refrigerated	7 days

New Specimen Stability		
Specimen	Temperature	Time
Urine	Frozen (preferred)	28 days
	Ambient	8 hours
	Refrigerated	72 hours

Questions

Contact Joshua Yang, Laboratory Resource Coordinator at 800-533-1710.